

Adjustable Stool

Thank you for purchasing this Croydex product. Please follow the fixing and care instructions to ensure your adjustable stool is fitted safely and retains its high quality finish. Please retain this leaflet for future reference.

This installation guide applies to the following product:
AP130222 - Small White Adjustable Stool

KEEP DIY TOOLS OUT OF THE REACH OF CHILDREN

Product Care

To retain the best quality finish, clean product regularly with a soft damp cloth. Do not use abrasive or chemical cleaners as these will damage the product. Dry the adjustable stool after use. Do not strike the product with hard or sharp objects.

Product Location

The adjustable stool must sit flat within the bathroom or within a shower tray. Curved surfaces of the bathtubs are not acceptable.

Cautionary Note

Please ensure all legs are equal length, the stool is level and all pins are secured in position.

Take care not to extend leg beyond its furthest extension. If the leg is extended beyond this point please note that spring tension will propel the pin from the stool, so take care when adjusting the height.

If the pin and spring are removed from the leg, re-insert these components as shown in Fig.A.

Please note that the spring and pin need to be compressed in order to re-insert the leg into the stool.



Fig.A

Fitting Instruction

1. Flip the adjustable stool to reveal its underside.
2. Insert the stool leg into the housing by compressing the small pin - See Fig.1
3. The small pin will 'Click' into the housing securing the leg in position - See Fig.1
4. Repeat this process for all four stool legs.
5. Once all four legs are secured, flip the stool into its useable position.



Fig.1

Additional Information

Maximum safe working load for the adjustable stool is 100kg

For details of the other items in the Croydex range please visit our website: www.croydex.com

Croydex Ltd, Central Way, Andover, Hants, SP10 5AW, UK
Tel: +44 (0) 1264 365881
Fax: +44 (0) 1264 356437
Email: info@croydex.co.uk