




### 1 SINGLE BATH STEP

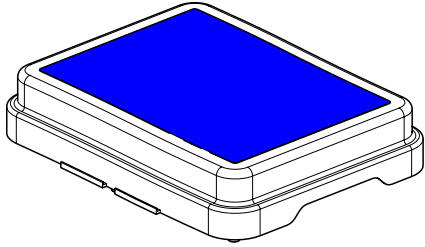


2 Insert the cover caps into the 4 cavities.

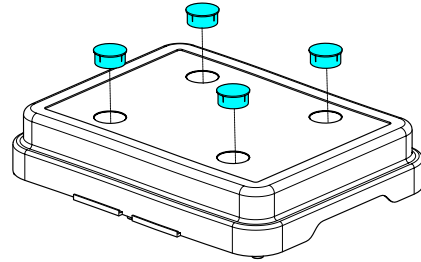

3 Peel the protective film from the foam pad.

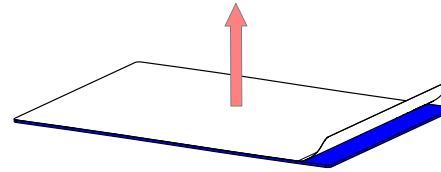


4 Place the foam pad on the bath step with the sticky side facing downwards. Apply even pressure across the step.

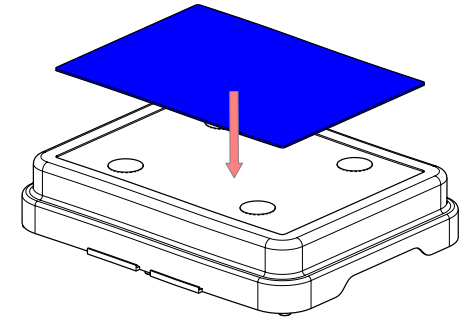


### 1 DOUBLE STACKED BATH STEP


Maximum 2 steps to be stacked.

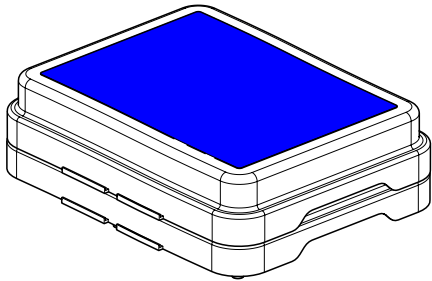


2 Place the cover caps into the 4 cavities on the top bath step. Then place that bath step on top of the other.

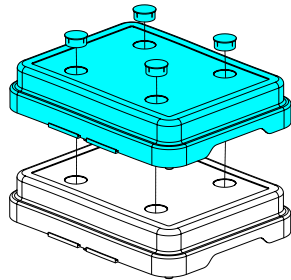


3 Peel the protective film from the foam pad.

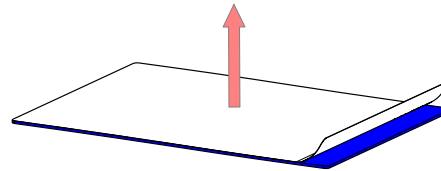


4 Place the foam pad on the bath step with the sticky side facing downwards. Apply even pressure across the step.

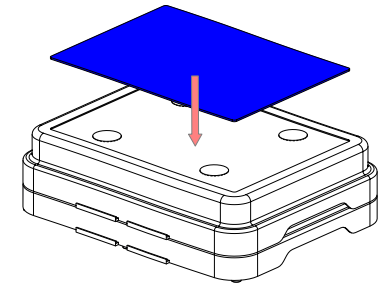

### 1 MULTIPLE BATH STEPS

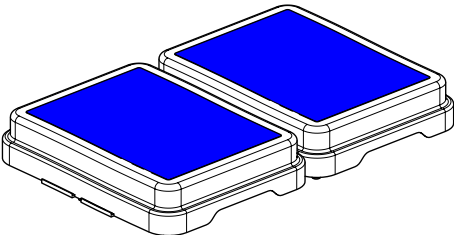

2 Secure the two or more bath steps by lifting the cavity on one step onto the corresponding hooks on the other.

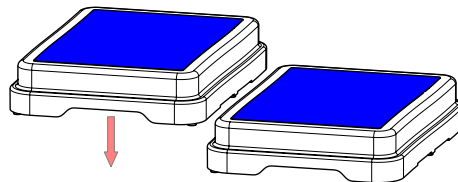

### SAFETY & CARE

1. To retain the best quality finish, clean product regularly with a soft damp cloth.
2. Do not use abrasive or chemical cleaners, as these will damage the product.
3. The maximum load for the product is 416lbs/189kg evenly distributed. **DO NOT EXCEED THE MAXIMUM LOAD.**
4. Please ensure that the product is placed on a flat surface.

